Electrophysiology adapters

INFORMATION FOR SAFETY AND PERFORMANCE

(EN)

1 INTENDED USE

PG922 adapters allow to connect equipment with 2mm socket to pacing leads with "touch proof" safety plugs conform to the EN 60601-1 regulation.

2 PRECAUTIONS AND WARNINGS

- Please, do open the package only after a careful check of the integrity of the pouches: for the Sterile Kit FIAB sterilization would be compromised. It is guaranteed until the expiry date indicated on the package if It has been correctly stored and it is not damaged. During the use, make sure that the adapter stays fixed to the equipment socket.
- If necessary (for example if a new positioning is needed), extract the plug so that the adapter remains fixed on the socket.

WARNING!

- For the fixing procedure of the adapters use cyanoacrylate adhesive dispenser provided by Fiab
- The fixing procedure of the adapters on the electronic device must be carried out by the competent technical department of the hospital or other adequately qualified personnel
- Carry on the procedure of fixing the adapters at least 1 hour before the connection to the lead.
- Apply the adhesive carefully. Before its application check the dispenser to assure that the aluminum seal has been removed and firmly secure the plastic cap.
- During the use of the adhesive be careful because skin and eyes can be glued in few seconds.
- Store away from children's reach.
- Don breathe the vapors.
- Carry on the procedure in a well aired environment.
- Don't cover the pins with the adhesive because it could lead to the loss of electrical contact.

STERILISATION

Sterile Kit FIAB are sterilised with EO gas. Do not reuse. Reusing the product may lead to: possible presence of biological residues that may cause cross-infections; alteration of materials; loss of initial functional features of product.

Other models are supplied **Non-Sterile**.

3 CONTRAINDICATION AND POTENTIAL COMPLICATIONS

No contraindications/complications are indicated if the product is used according to the intended destination.

4 WASTE DISPOSAL

Waste coming from health facilities must be disposed of according to the regulation in force.

Rev.	Revision Status / Description	Date
A	First Issued	13/01/2025

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