Single use sterile electrocautery Information for Safety and Performance (EN)

1. PRODUCT DESCRIPTION

Electrocautery, sterile packaged disposable device powered by built-in alkaline batteries. By pressing the activation button, the device cauterizes by placing the incandescent tip on the patient.

2. INDICATIONS AND USERS

The devices must be used by qualified personnel.

Open the package and place the cautery in a sterile field.

After checking the integrity of the package, check the body of the cautery. In the presence of damage or visible defects, do not use the product and return it to FIAB.

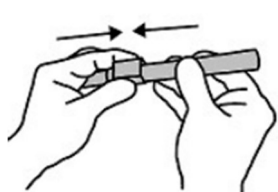
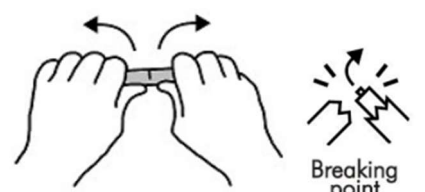
Remove the protective cap being careful not to damage the tip.

Before use, check the function as follows:

- press the activation key keeping the device away from people and things;
- for HTC models check that the tip becomes incandescent;
- for LTC models, touch a sterile gauze wet with water with the tip, verifying that 2-3 seconds after activation, evaporation of water from the gauze can be felt.

During use on a patient, perform short (2-3 seconds) and repeated activations by pushing the switch. After use, always replace the protective cap.

WASTE DISPOSAL: The product contains batteries, remove them as indicated, before disposing of the device. Waste coming from health facilities must be disposed of according to the regulation in force. The device may be contaminated with blood and must be disinfected before being discarded. In order to avoid accidental switching on (if the batteries have not been removed), cut the tip wire with pliers or scissors and always reinsert the protective cap

		
<p>Close the cautery with the stopper.</p>	<p>Grip the cautery as directed. Make sure that the part of the shaft to be broken is facing outwards. Apply a suitable pressure until the shaft breaks.</p>	<p>Remove the batteries taking care not be injured from sharp residues. Dispose of the batteries according to the regulations in force. Dispose of the remaining parts as contaminated waste.</p>

3. CONTRAINDICATIONS

The FIAB Electrocauterics are not suitable for cauterizing heavy bleeding.

4. INTENDED USE & ESSENTIAL PERFORMANCE

The FIAB Single use Sterile Electrocautery are designed for the cauterization of tissues and small blood vessels during surgical procedures.

5. CLINICAL BENEFIT

- Cauterization without connecting the patient to an electric circuit.
- Cauterization of patients carrying implantable devices sensitive to electromagnetic interferences.

6. STERILITY

WARNINGS:

The product is sterile in unopened and undamaged package. STERILIZED BY ETHYLENE OXIDE. DO NOT RESTERILIZE. Resterilization could change the physical characteristics of the material and should not be attempted.

Disposable product, DO NOT REUSE - Reuse may result in:

- Cross infections;
- Alterations of materials;
- The loss of the initial functional characteristics of the product.

Sterility is not guaranteed if the package has been opened and/or damaged.

Do not use the device after the expiration date written on the package.

7. PRECAUTIONS AND WARNINGS

WARNINGS:

Contains batteries, do not throw into fire.

Do not use in the presence of flammable material.

Rev.	Revision Status / Description	Date
A	First Issue	07/12/2022
B	Update §2	06/02/2025

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