PENCIL AND ELECTRODES FOR ELECTROSURGERY

INFORMATION FOR SAFETY AND PERFORMANCES

INDICATIONS

Clinical procedures requiring monopolar electrosurgery.

INTENDED PURPOSE

Conduct electrical energy from the monopolar electrosurgical generator to the surgical site in order to achieve the desired surgical effect.

INTENDED USERS

The devices are to be used by trained healthcare professionals in electrosurgical procedures.

PATIENT POPULATION

Given the multiplicity of clinical situations requiring electrosurgery, a patient population that is very heterogeneous in terms of age, weight, health and clinical conditions may be suitable for such procedures.

CLINICAL BENEFIT

Enable cutting and coagulation of tissue during monopolar electrosurgical procedures.

NB Pacemaker or other active implants may be damaged by an electrosurgical current. Do not subject a patient with a pacemaker other active implants to an electrosurgical current without having previously consulted a cardiologist or other qualified person.

PRECAUTIONS AND WARNINGS

- STERILISE BEFORE USE those products which are not pre-sterilised.
- After checking the integrity of the package, please check its content; in case of visible damages or defects, do not use the product and return it to FIAB.
- The distal part of the hand-switch pencil can hold all models of electrodes (ACTIVE PART) and extensions having a shank diameter of 2.38 mm (3/32") or 4 mm depending on the model. Be sure that the electrode or extension (if present) is firmly inserted in the pencil, and that the insulating coupling of the electrode and extension (if present) is intact.
- Always check the functioning before use. Discard defective products.
- Be sure that the connection of the hand-switch pencil's plug with the generator is firm, and that the metallic part of the connector is completely inserted.
- Keep the products away from inflammable materials: their accidental activation and heat could cause fire.
- When the electrosurgical accessories are not in use, they must be stored in an isolated and secure position. for examole in a holster.
- For the use of the generator, pencils, neutral electrodes, cables and other accessories, please refer to the instructions for use supplied by the manufacturer.
- **Caution:** Risk of injury from sharp edges.

MAXIMUM APPLICABLE VOLTAGE 9kVpp

RESIDUAL RISKS AND POSSIBLE CAUSES OF BURNS

Residual risks associated with cutting and coagulation procedures with the electrosurgical pencil: wound complications (infections, inflammation), burns, interference with active implants.

Burns may be caused by a high density of current in the patient's tissues or by the heating of inflammable fluids or gases. Possible causes are:

- The patient has inadvertently been placed in contact with electrically-conductive parts.
- A direct contact has occurred between the cables and the patient's skin, which has caused a capacitive effect.
- The combustion of inflammable disinfectant agents.
- The combustion of inflammable narcotic gases.

CONTRAINDICATIONS

The products must not be utilised if:

- There is visible damage to the electrode or extension, to its insulation, to the body and/or connector of the pencil, or to the connection cable.
- The preliminary operational check gave negative results.

VALID ONLY FOR DISPOSABLE PENCIL

The product is disposable and sterilised with Ethylene Oxide gas. Do not reuse. Reusing the product may lead to:

- Possible presence of biological residues that may cause cross-infections.
- Alteration of materials.
- Loss of initial functional features of product.

Sterility is not guaranteed if the package has been opened and/or damaged.

Do not use after the expiration date. Sterility is not guaranteed.

VALID ONLY FOR REUSABLE PENCIL

PROCESSING - Process accessory device through a cleaning and sterilization cycle prior to initial or subsequent use per instructions below.

- Do not press the push-buttons during the washing operation.
- Do not make forced entry of liquids or air inside the receptacle of the tip.
- Do not use ultrasound baths.
- The use of mechanical washing systems may reduce the product's life.
- Do not use abrasive mechanical systems on the insulation of the electrode or extension to remove biological residue, as such systems could damage it.

CLEANING - Before proceeding with the cleaning and disinfection operations, remove the protective cap (if present) from the electrode; remove the electrode and extension (if present) from the tip of the pencil. Treat the entire device surface with a cloth soaked in a decontamination solution. Let the solution act for the required time set out on its technical datasheet. Rinse the device with a water soaked cloth. Decontaminating products based on surfactants and enzymes are recommended. Use the product according to the dosage indicated in the manual.

Thermal disinfection - If thermal disinfection is performed with an automatic instrument washer, the washer disinfectant must comply the requirements of the ISO 15883 series.

STERILISATION - The devices in object of present instructions can be sterilised in a steam autoclave.

NB The sterilisation process used must conform with existing regulations.

- Validation of the process is the responsibility of the user.
- Do not sterilise the pencil when the electrode or extension is inserted in the tip.
- In repackaging, use adequate protection systems for the tip in order to avoid perforation of the pouch, with the consequent loss of sterility.

- Package the products in suitable pouches indicated for the sterilisation process.
- Avoid wrapping that is too tight and sharply twisting of the cord.
- Allow product to cool at least 30 minutes before use.

Validated temperatures and times for steam autoclave are as follows:

Cycle	Pre-conditioning		Exposure	Post-conditioning	
	Pre-vacuum	Heating		Drying	Ventilation
134°C (273°F)	20min	6min	134°C (273°F) 3,1 bar for 5min	24min	2min
121°C (250°F)	18min	5min	121°C (250°F) 2,1 bar for 20min	10min	2min

ATTENTION - Should the products come in contact with the metal supports within the autoclave, their life may be shortened. It is therefore necessary the use of appropriate means spacing or wrapping as gauze.

NB The pencils can NOT be sterilised by 137°C (279 °F) cycle.

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